

Feel free to contact us with any inquiries;  
our technicians are ready to assist you when  
you call.



@Nimrodental



@NimrodTal



@Nimrodental



@Nimrod Tal



@Nimrodental



02073876930



contact@nimrodental.co.uk




## Bonding Procedure for Tooth Coloured Milled Bonded Retainer



## Please read the following tips to help you during the bonding process.

- To position the lingual bar correctly, place the caps over the premolars.
- Placing the composite as shown on the diagram is crucial.
- Once the composite has been bonded, remove the positioning caps by cutting them away with either a disc or a fissure.

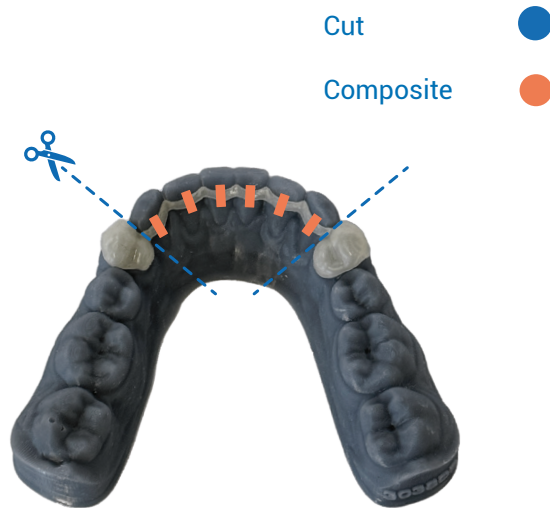

## Tips on bonding fixed retainers:

- Ensure the retainer sits passively on the patient's teeth.
- For upper retainers, check the occlusion with the lower incisors.
- Roughen the surfaces to ensure better adhesion.
- Keep the bonded areas dry, moisture control is crucial!
- Apply an adequate amount of composite and light cure.
- Cut away the jig and smooth the edges of the wire.
- Teach the patient the proper techniques for cleaning it.



Scan for a video guide